
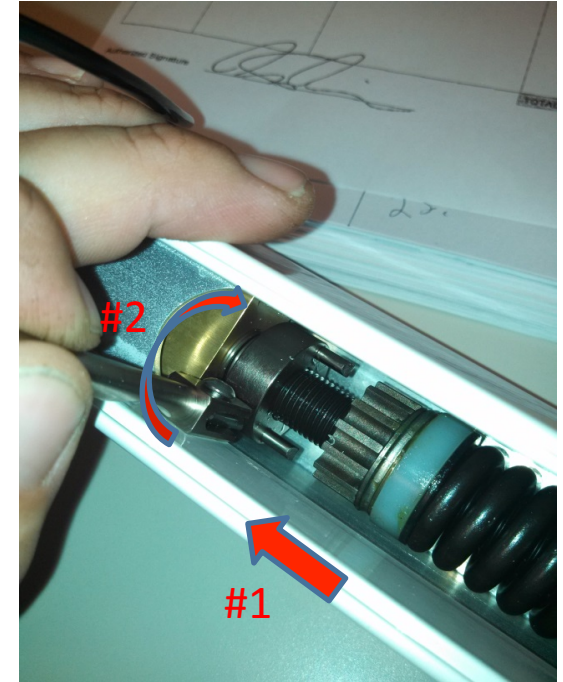


## Topic: Adjustments

### Problem: Flex arm is not properly counter balanced

The pictures below show the bottom of the flex arm with the cover slid out of the way for clarity.

**To increase tension** - Turn the lever attached inside the flex arm in the direction indicated. Then slide it free of the toothed nut, turn it back the opposite direction and re engage it with the toothed nut. Repeat process until the arm is properly tensioned. **\*\*To reduce tension simply turn the toothed nut in the opposite direction\*\***

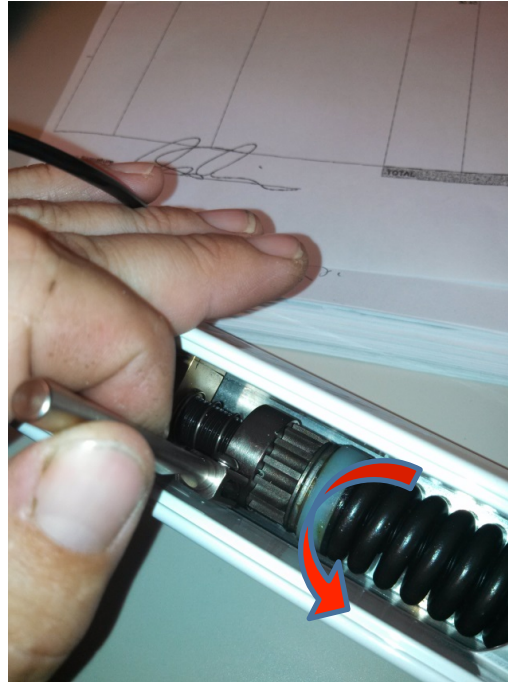

If the arm needs a lot of adjustment it is sometimes easier to use the following method.

Disengage the lever arm from the toothed nut.

Then using a screwdriver lever the toothed nut in the proper direction to tighten or loosen the spring assembly. Use caution to avoid damaging painted surfaces that will be exposed.

